

Give your facility a competitive advantage, and give family members peace of mind with our monitoring system.



Hello Nurse is always on, monitoring for specific events on monitored residents.

The Hello Nurse system works independent of facilities IT systems and does not require the use of WIFI. Caregivers can respond immediately to wet or high fall-risk residents while the software logs response times, creating a much more efficient staff while improving the quality of care and lowering costs.



“As part of our continuous performance improvement initiatives, We identified resident falls as a quality measure we wished to improve. Since installing Hello Nurse, we have seen a reduction in falls of the residents who have utilized this monitoring system. We are very pleased with the Hello Nurse system.”

Healthcare Administrator, Providence Point

Providence
POINT

Where Wellness Meets Wireless

No other system provides fall management, repositioning of bed bound residents and incontinence monitoring while electronically capturing the data.



Wet Alert — Incontinence Monitoring

When connected to a Hello Nurse wireless transmitter, specially designed Hello Nurse sensors, designed to fit any and all briefs or pads, detect moisture and send an alert wirelessly to the Hello Nurse central monitoring screen. Caregivers are alerted instantly, which allows them to respond quickly, reducing residents' chances of contracting dermatitis and infection — preventing the need for additional care and treatment.



Pressure Alert — Fall Prevention and Turn Monitoring

Specially designed for fall management purposes, Hello Nurse pressure pads detect the movement of a high fall risk resident as they may be attempting to stand and send an alert wirelessly to the Hello Nurse central monitoring screen. The Hello Nurse system is the only system of its kind that notifies caregivers using wireless pressure pads BEFORE a fall can occur. Immediate attention can then be provided and many incidents can be prevented. Bed-bound or invalid residents benefit tremendously from regularly being repositioned, preventing pressure-induced skin breakdown. Hello Nurse alerts can be set to signal caregivers to turn residents every two hours or on a custom schedule.



Nurse Call Alert — Pushbutton Nurse Paging

Hello Nurse's wireless transmitters are designed with a Nurse Call push button. When the button is pressed, the transmitter will send an alert to the Hello Nurse central monitoring screen. For rooms that are without Nurse Call functions, Hello Nurse's transmitters can be used to create an instant network of Nurse Call nodes. Since Hello Nurse is wireless, there is no need for expensive renovation to install the Nurse Call system.



Repositioning Alert — Scheduled Reminders

The system sends an alert to the caregiver's pager letting them know it's time to reposition a bedbound resident. When the caregiver repositions the patient and resets the transmitter, the event is logged in the software for a hard copy record or compliance.

Gain valuable insight to your staff's efficiency and spot trends more easily, giving you an unprecedented view of your performance.

1

Simple Installation

For fall prevention, one pressure pad for chairs or beds. For incontinence monitoring, a sticker is placed on the outside of any kind of brief. Each Receiver (Nurse Station) can handle up to 30 Residents. You can expand that capacity by adding repeaters.

2

Instant Alerts

When the alarm is triggered by moisture or by a resident attempting to get up off of a bed or chair, an alarm is sent to a pager or cell phone with the alarm type, the resident's name and room number allowing the caregiver to quickly respond.

3

Easy & Accessible

The Hello Nurse receiver alerts caregivers through the attached paging system while simultaneously recording and logging all alert and event data in the Hello Nurse patented software. The reporting can be used for staff performance, family meetings, determining incontinence patterns and deciding if residents need the use of different equipment.

4

Accurate Attention

Caregivers receive an instantaneous page on their pagers with resident name, room number and type of alarm that was triggered, allowing for more accurate and quicker attention upon response.

5

Assured Response

The caregiver can respond quickly to the event and can only clear the alarm by resetting the transmitter.

Try Hello Nurse Wireless Patient Monitoring and alert technology risk-free* for 30 days in your facilities.

Learn more at:

hellonursemed.com

Call us at:

949.341.0676

Email us at:

info@hellonursemed.com

*Pre-approval required

Help is on the way.